



Fast, Easy 4-minute Screening Test For the Detection of Alcohol

by



TM

ISO 13485 Registered

DOT APPROVED

FDA CLEARED

CLIA WAIVED

The most cost-effective DOT-Approved saliva alcohol screening test on the market, Alco-Screen 02 is ideal for your DOT mandated testing program.



MADE IN USA



## ADVANTAGES

Economical

Convenient  
use anywhere, anytime

Quick 4-minute test

DOT Approved

FDA Cleared

CLIA Waived

Over the Counter use

## USES

DOT mandated  
testing

Workplace testing


Alcohol treatment  
programs

Criminal justice  
testing


Emergency room  
admission testing



## PROCEDURE

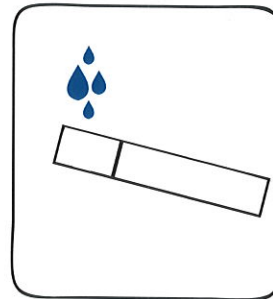

1. Apply saliva directly to the test pad.

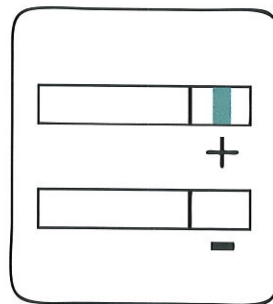

2. At four minutes, observe the reagent pad. A distinct colored line across the reagent pad indicates a positive result. A reagent pad with no distinct line indicates a negative result.

MADE IN USA

56620 Rev 1

